

Quarterly Test Sheet B - Washer Disinfectors (SHTM 01-05)

To be filled in with Quarterly Test Sheet A (FAC414-076.03) to complete a quarterly test. Tests to be carried out in accordance with Scottish Health Technical Memorandum (SHTM) 01-05 Part B.

Table 1 - Record details

Hospital/ Location		Week beginning		Week number	
Department		Ref number		Serial number	

Cleaning Efficacy Test

Table 2 - Cleaning efficacy test. Chamber walls, load carrier and reference load. Abort cycle after wash stage. Each cycle available should be tested

Load type	Cycle selected	Cycle number	Result	Comments
			Pass/ Fail	
			Pass/ Fail	
			Pass/ Fail	
			Pass/ Fail	
			Pass/ Fail	
			Pass/ Fail	
			Pass/ Fail	
			Pass/ Fail	
			Pass/ Fail	
Type of test soil used:		Batch number and expiry date:		
Amount of test soil applied:		Drying time:		
Additional comments:				

Verification of Calibration/ Thermometric Test for Thermal Disinfection

Table 3 - Verification of calibration/ thermometric test for thermal disinfection. *The items in slowest and fastest positions to disinfection temp. may be the same as top, middle or bottom carrier items

Cycle selected:		Cycle number:	
Details of load:			
Time/temperature required for thermal disinfection (tick box)			
70-75°C for 100 min		80-85°C for 10 min	
		90-95°C for 1 min	
Temperature sensor type/ location	Load item if applicable	Disinfect hold time	Temp at start of stage (°C)
			Temp at end of stage (°C)
			Within 2°C of measured control sensor
			Time/ temp within limits
Machine control (sump)			Yes/ No
Measured sensor by above			Pass/ Fail
Machine recorded/ printed			Yes/ No
Measured sensor by above			Pass/ Fail
Top load carrier			Pass/ Fail
Middle load carrier			Pass/ Fail
Bottom load carrier			Pass/ Fail
Slowest region*			Pass/ Fail
Fastest region*			Pass/ Fail
Fluctuation of measured load temps within 2°C?	Yes/ No	Measured load temps within 4°C?	Yes/ No

Faults Record

Table 4 - Table to record faults (new or existing). Also enter in plant history record in FAC414-076.01

Fault comments

Test Outcome

Table 5 - Table to detail tested by, date completed, and test results

Test result	Satisfactory/ Unsatisfactory	Washer disinfectors fit for use?	Fit/ Unfit
Competent Person (Decontamination) (CP(D))		User	
Date		Date	