

Daily/ Weekly Test Sheet - Washer Disinfector (SHTM 01-05)

Tests to be carried out in accordance with Scottish Health Technical Memorandum (SHTM) 01-05 Part B.

Table 1 - Record details

Hospital/ Location		Week beginning		Week number	
Department		Ref number		Serial number	

Daily Checks

Table 2 - Daily checks

Day	Filter and strainers clean	Spray arms/ nozzles (on machine)	Spray arms/ nozzles (load carriers)	Detergent levels
Monday				
Tuesday				
Wednesday				
Thursday				
Friday				
Saturday				
Sunday				

Automatic Control Test

Table 3 - Automatic control test - Use a stopwatch for timings where no independent recorder is fitted, and information cannot be noted via Washer Disinfectors (WD) printout. Where multiple pre-set cycles are available and used, the cycle selected for the test should be rotated among those used. This should include a WD self-disinfection cycle if available

Cycle selected:				Cycle count number:			
Details of load:							
Time/ temperature required for thermal disinfection (tick box)							
70-75°C for 100 min		80-85°C for 10 min		90-95°C for 1 min			
Start t=0	Time stage starts	Time stage ends	Time for stage	Indicated temperature mid stage (°C)	Recorded temperature mid stage	Water pump pressure mid stage	Time/ temp within limits
Pre-rinse							Pass/ fail
Pre-wash							Pass/ fail
Wash							Pass/ fail
Rinse							Pass/ fail
Disinfect / rinse							Pass/ fail
Time above °C							Pass/ fail
Drying							Pass/ fail
Cooling							Pass/ fail
Total cycle time							Pass/ fail
Visual display of cycle complete (tick)				Cycle variables within validation limits (tick)			
Disinfection time/ temperature within limits			Yes/ No	Door(s) remain locked until cycle complete (tick)			

Weekly Checks

Table 4 - Weekly checks

Condition of door/ lid seal and interlocks:		Condition of bath, load carrier/ basket:	
Protein residue test method used:		Batch number/ Expiry date:	
Instrument(s) swabbed:		Result:	Pass/ Fail
Additional comments:			

Faults Record

Table 5 - Table to record faults (new or existing). Also enter in plant history record in FAC414-076.01

Fault comments

Test Outcome

Table 6 - Table to record test outcome

Test result	Satisfactory/ Unsatisfactory	Washer disinfectant fit for use?	Fit/ Unfit
CP (D)		User	
Date		Date	